



Critical considerations for the design of a safe, performing and compliant medical package

Nicole Kaller
Packaging Engineer EMEA
DuPont Medical and Pharmaceutical Protection

Tuesday, 28 February 2017



The miracles of science™



Learning points

- Basic functions of a package
- Peel pouches, headerbags, thermoformed blisters – what needs to be considered?
- Temperature and pressure variations during transport
- Product families

Basic Functions of a Medical Package

- Product protection
 - Microbial barrier
 - Physical protection from damage
- Compatibility with the sterilization method
- Maintenance of sterility and integrity until use
- Easy opening and aseptic presentation
- Identification of the product (printing, labeling)



1. Peel Pouches

Design tips


- Allow ample room between chevron tip and edge of pouch



1. Peel Pouches – cont'd

Folding should be avoided


- Can form abrasion points with the carton
- Scuffing can cause holes in the package – made of Paper, Nylon, Tyvek[®],...

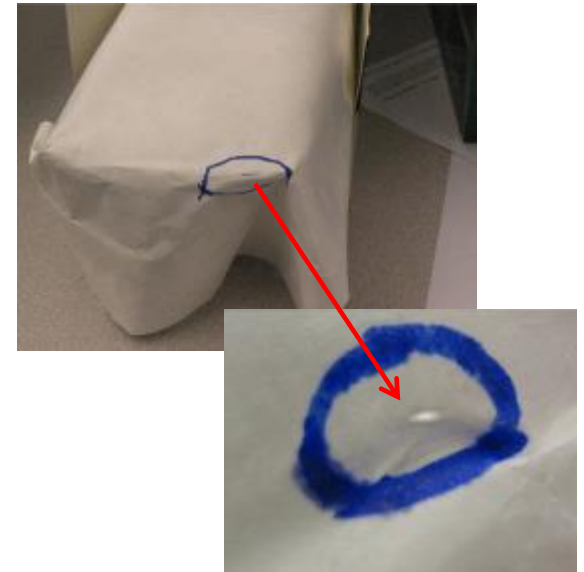

1. Peel Pouches – cont'd

Folding can cause a sheet inner delamination (Tyvek®) or even material break (cellulosic material)

- Inner delamination can lead to false positives during integrity testing
- Microbial barrier is not compromised with Tyvek®
- Be careful with other porous materials!

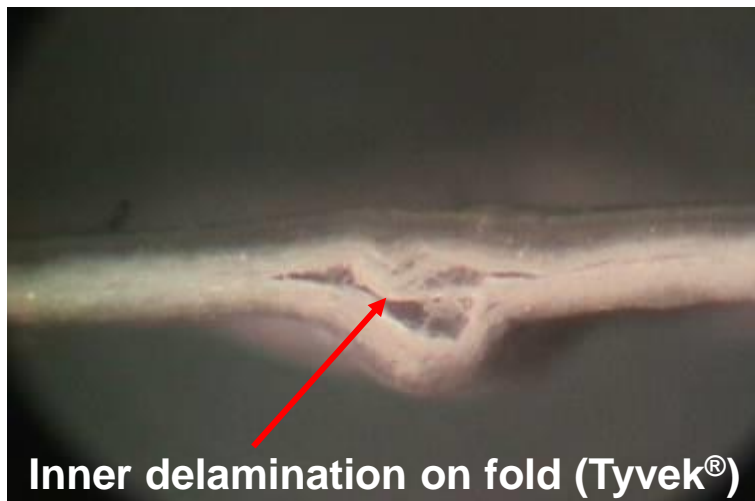


2. Header bags



Design tips

- Design the opening large enough to allow the product to be aseptically presented.
- Association of periOperative Registered Nurses (AORN)...
 - ...Considers the inner edge of the heat seal as the line separating sterile from non-sterile

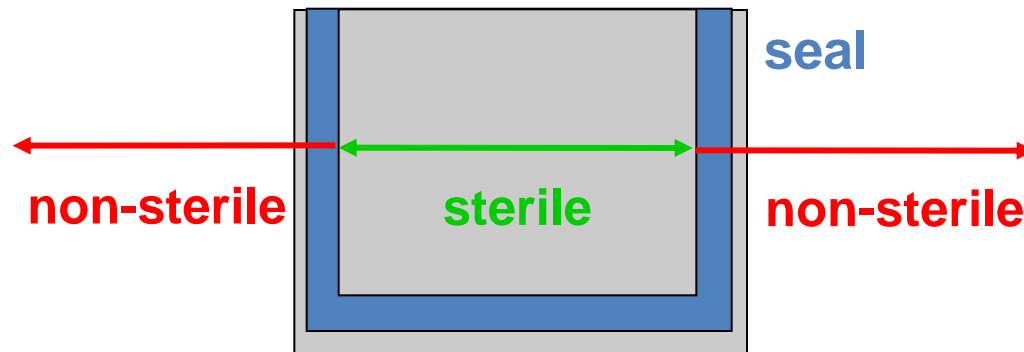


3. Thermoformed blisters

A faulty lidding can cause **delamination** or **tear** during opening:

Risk:

Compromized Aseptic Presentation


→ **Contaminated Device**

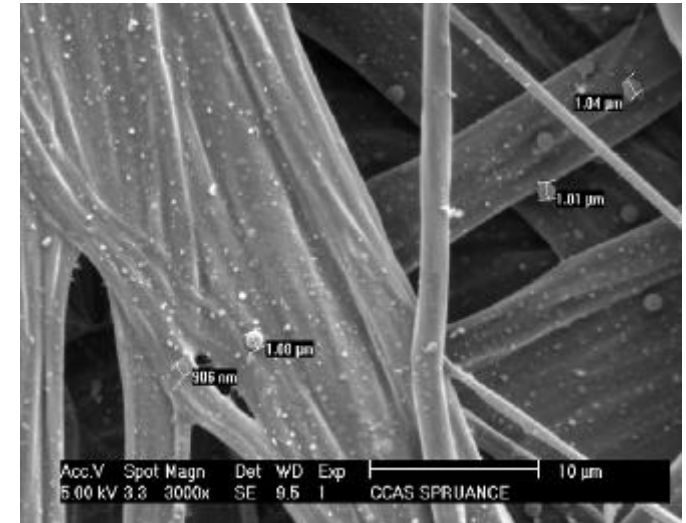

3. Thermoformed blisters

– Triggers of delamination and tear

1. Misplacement of the porous lid or cut through blister seal

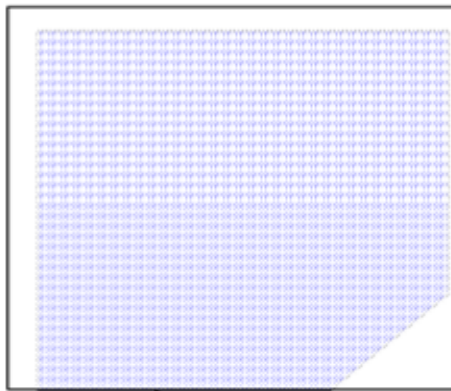


Possible Fiber Tear


2. Oversealing / Transparentization



3. Ragged edge caused by faulty knives

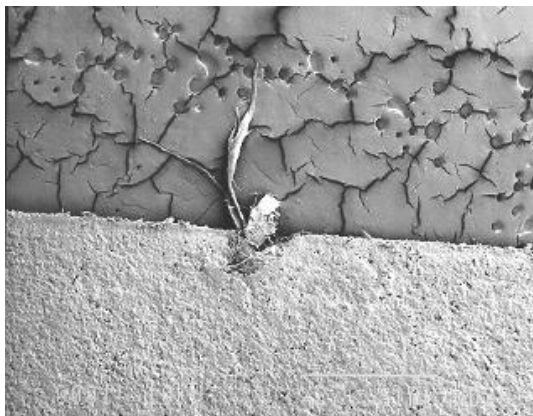


4. Non-adapted peel system

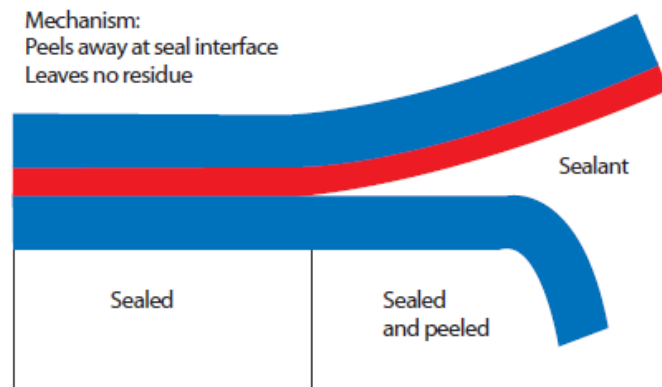


3. Thermoformed blisters - Preformed

3-Layer TYVEK® STRUCTURE

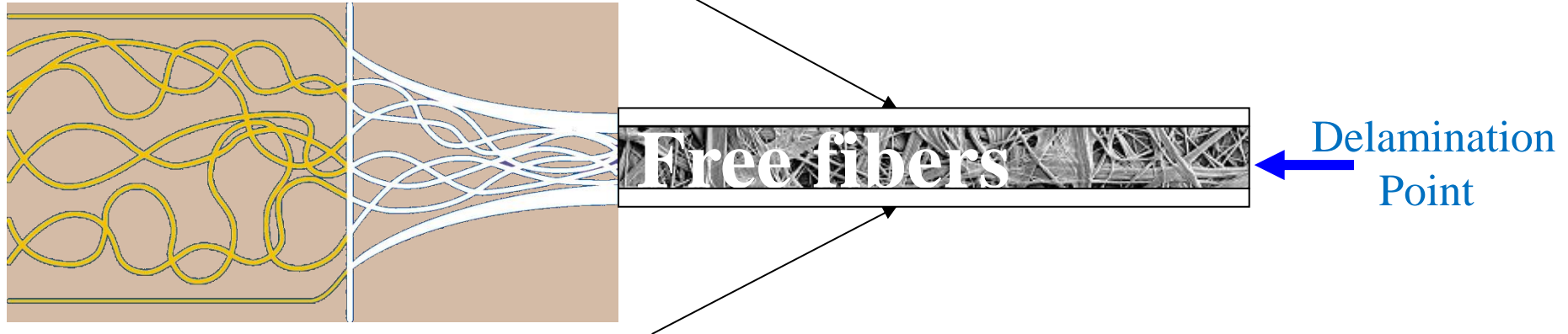

Solution: Unsealed gap

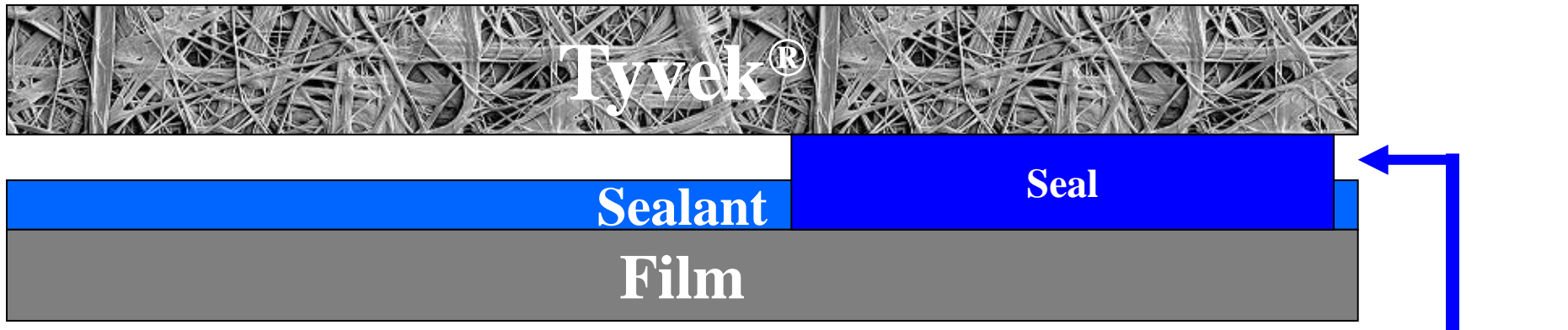

Unsealed gap
> 0,5 mm





3. Thermoformed blisters - Preformed

Solution 2: Blister geometry




- Eliminates lid placement issues
- Condition: Sufficient size of the lid

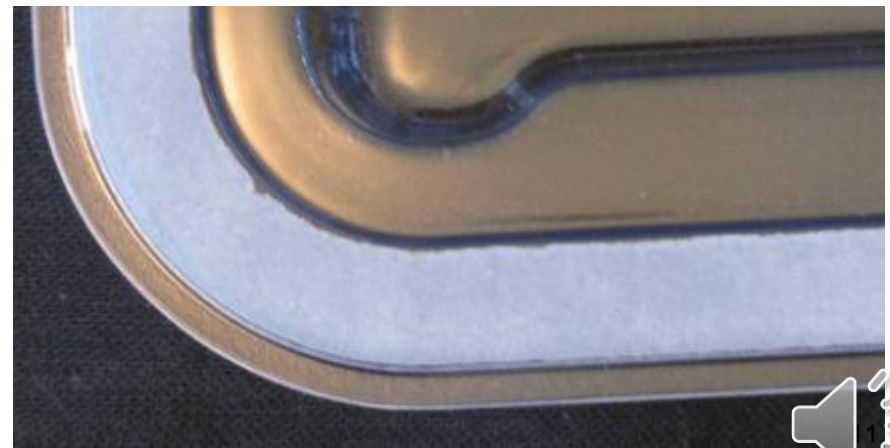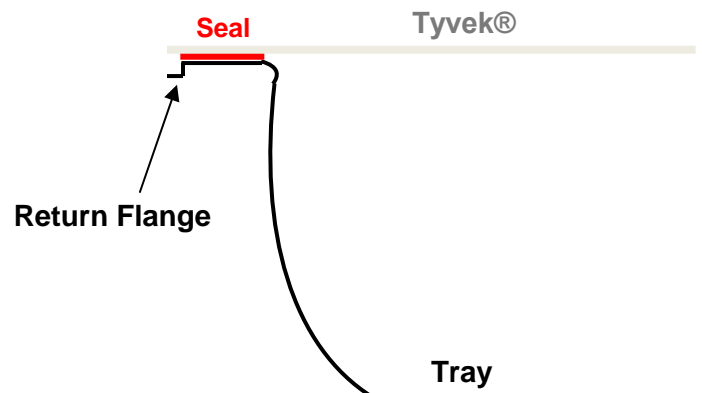

3. Thermoformed blisters – Form-Fill-Seal

Solution: Customized sealing tool

Keep edge unsealed

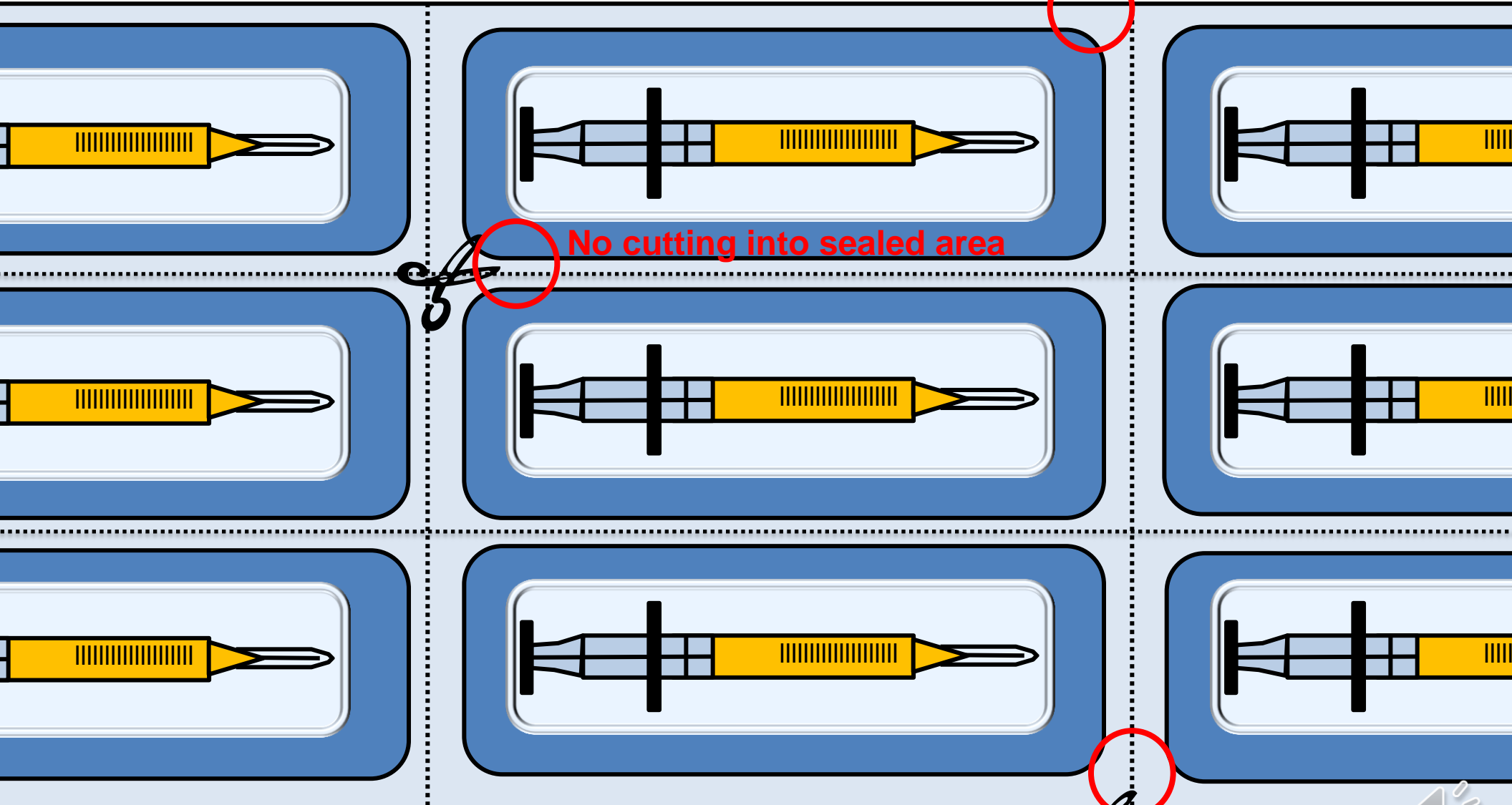

No cutting into sealed area

No cutting into sealed area


Oversealing and transparentization

Note:

- To be avoided for **peelable** seals
- Not visible when using paper
- Transparentization is a result of melting Tyvek®

How to avoid:

- Reduce sealing energy transfer (e.g. Temp., Time, Pressure)
- Add Teflon® sheet to the heat seal platen
- Check for “hot spots” on the tool (e.g. pressure-/ thermo-sensitive paper)
- Check durometer/stiffness of rubber matting material (shore 70 for Tyvek®, riffled pattern)

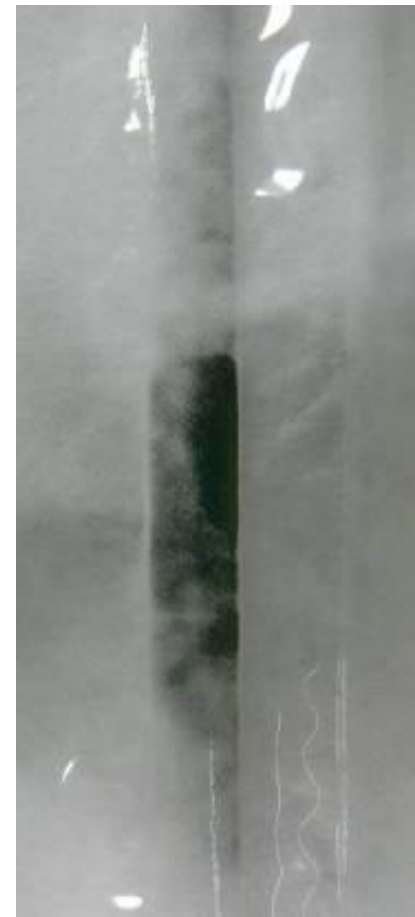

Temperature and pressure variations during transport

- Results in pressure differences inside package vs. outside package - non breathable materials can't adjust to changes in air pressure
- An excessive pressure can damage the packaging



Product families

– one investment to optimize working effort

One proven packaging configuration for different devices,
using

the same reliable and best performing materials,
has many benefits:

- Less machine downtime and start up time necessary
- Less process waste
- Less validation and re-validation effort
- Easier process control



→ **Higher productivity and lower costs**

Conclusion


A good packaging solution, well thought through, properly tested and validated, will minimize your risks and will provide you PEACE OF MIND for many years!



Technical documentation – DuPont™ Tyvek®

- DuPont™ Technical Reference Guide for Medical and Pharmaceutical Packaging
- Dupont™ Tyvek® Compliance to ISO 11607-1:2006
- DuPont™ Tyvek® Graphics Technical Handbook

www.MedicalPackaging.DuPont.com

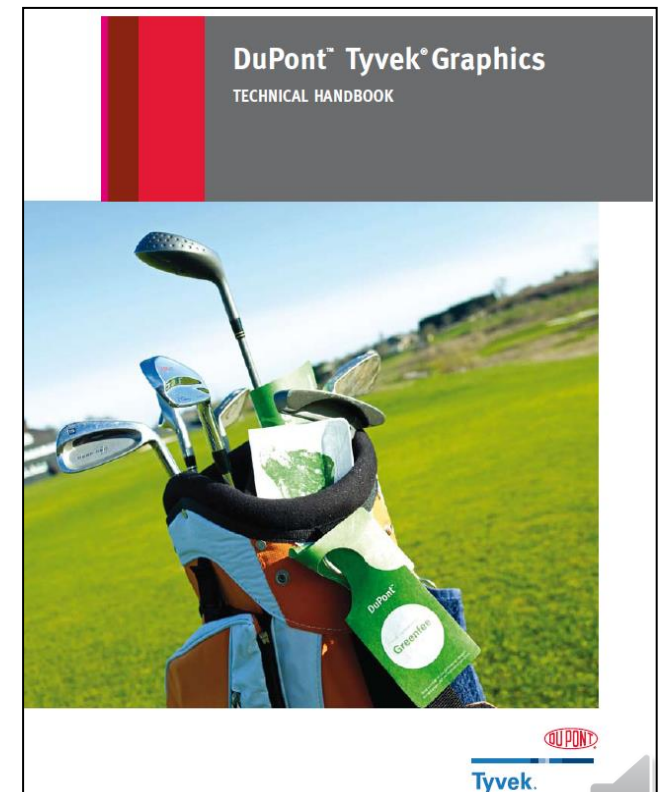

DuPont Medical Packaging

More Information . . .

www.MedicalPackaging.DuPont.com

■ or call DuPont in Luxembourg

+352 3666 – ext.

Nicole Kaller: 7032

Eric Schmohl: 5890

Thierry Wagner: 5330

Helmut Scheckenbach: +49 6102 18 1806



THANK YOU FOR YOUR INTEREST!

Disclaimer

This information corresponds to our current knowledge on the subject and may be subject to revision as new knowledge becomes available.

It is your responsibility to investigate other sources of information on this issue that more appropriately addresses your product and its intended use. This information is not intended for use by you or others in advertising, promotion, publication or any other commercial use. **DUPONT MAKES NO WARRANTIES OF ANY KIND REGARDING THIS INFORMATION AND ASSUMES NO LIABILITY WHATSOEVER IN CONNECTION WITH ANY USE OF THIS INFORMATION.** This information is not a license to operate under, or intended to suggest infringement of, any existing trademarks or patents.

© 2013 DuPont. All rights reserved. The DuPont Oval Logo, The miracles of science™, and Tyvek® are registered trademarks or trademarks of E.I. du Pont de Nemours and Company or its affiliates.